SEAC[®]

MANUALE D'USO
USER MANUAL

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1 Safety regulations

1.1 Correct use

Full-face masks are to be used with P3 respiratory protection masks that are EN 143:2007 certified.

This manual must be read and followed carefully when using the product. In particular, the safety instructions as well as information relating to the use and operation of the product must be read and followed carefully. Furthermore, for safe use, the current national regulations must be respected.

When using full face masks with a respiratory protection filter, the following conditions must be met:

- It is essential to know the environmental conditions (in particular the nature and concentration of the harmful substances present) and to use suitable respiratory protection filters.
- Use filtering equipment only if the air does not represent any immediate health hazard.
- Replace the respiratory protection filter every 24 hours of use, to ensure proper antiviral protection. The new filter must be a P3 type and EN 143:2007 certified. We recommend the DIRIN 230 RD P3 model or equivalent.

• **WARNING!**

If used correctly, the product can be considered a respiratory protection device against pathogens such as germs or viruses. Incorrect use or maintenance of the device can compromise the correct functioning of the device and consequently endanger the health of the user.

Before use, it is necessary to check the correct functioning of the product. The product cannot be used if the functional test has not been successful, if there are damages, or if non-original spare parts have been used.

The mask cannot be used by different operators without first having replaced the filter and its cleaning and disinfection.

Any use different or not in compliance to these specifications will be considered non-compliant. This especially applies to unauthorized modifications to the product.

• **ATTENTION!**

Use of this PPE is NOT anticipated in environments with an explosive atmosphere or in the presence of open flames; it is expected to be used in environments without significant heat radiation.

- ATTENTION!

The product must absolutely not be used by people with respiratory and / or lung problems and in any case with reduced respiratory capacity, nor should it be used if you are not 100% of your psychophysical conditions. Failure to observe these warnings could cause serious accidents.

- ATTENTION!

If you begin to feel unwell, breathless, dizzy, or like you are not getting enough air when using the mask, first move away from the potentially contaminated area. Do not pull off the mask and breathe deeply until you are outside.

1.2 Liability information

Seac Sub declines all responsibility in cases if the product has been used in an inappropriate or non-compliant way. The choice and use of this product are the sole responsibility of the individual operator.

The claims for compensation for product liability, and the guarantees offered by Seac Sub regarding the product will not be valid if it is not used, checked and maintained according to the instructions given in this manual.

2 Description

The body of the mask is made using a special compound based on polycarbonate and anallergic silicone that ensures that the mask adapts comfortably and snugly to the face and provides an excellent seal.

The inhaled air flows from the filter of the mask, beyond the inhalation valve, towards the inside of the viewer [thus avoiding fogging], then passes through the internal valves and reaches the oro-nasal area.

The exhaled air passes directly to the ambient air through the exhalation valves.

Certification

The CE marking on the product refers only to the use as DPI cat.I for snorkeling use: it protects the eyes from contact with water.

3 Use

The mask must be carried in its original packaging.

• ATTENTION!

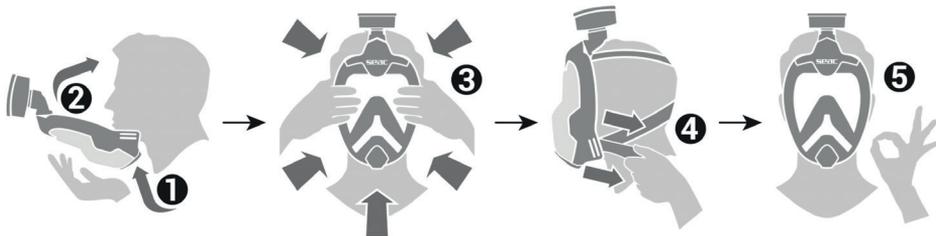
Make sure that the mask's skirt adheres to the user's forehead and face. There must be no hair between the sealing edge of the mask and the user's skin, otherwise the mask may have leakage. This risk exists, for example, even when the wearer of the mask has a beard or deep scars in the sealing area.

3.1 Put the mask on

• ATTENTION!

Normal glasses cannot be worn under the mask.

- 1) – 2) Widen the elastic strap and position the mask correctly on the face.
- 3) Let the skirt adhere to the entire surface of the face.
- 4) Pass the elastic strap over the head, let it adhere to the neck and adjust it to the sides of the face so that the mask is well fixed.
- 5) Check to have correctly performed the previous steps, and to wear the mask correctly.

3.2 Sealing test

Before using the mask, make sure you ALWAYS perform a sealing test, to be sure the mask perfectly sealed on your face.

Perform the sealing test after replacing the respiratory protection filter

- 1) Plug the air inlet hole on the filter with your hand.
- 2) Test the sealing by inhaling and exhaling. During this step, when you inhale, there must be negative pressure and there must be no inflow of air.
- 3) If necessary, tighten the straps.

If the sealing test fails, the mask should not be used.

3.3 Take the mask off

- 1) Losen the strap.
- 2) Pass the elastic strap over the head, towards the face.
- 3) Take the mask off the face.

4 Cleaning and disinfection

· WARNING!

The respiratory protection filter must be replaced every 24 hours of use. Failure to observe this rule seriously endangers the health of the user.

Cleaning and disinfection of the masks must be performed after each use.

· ATTENTION!

Do not use cleaners containing hydrocarbons, solvents [for example, nitro thinner], or cleaners containing aggressive surfactants.

Clean parts must not be dried with radiant heat [e.g. sun, radiators].

After drying, keep the mask away from direct sunlight.

Perform a sealing test at the end of the cleaning, disinfection and every time the membranes of the valves and / or the respiration protection filter are replaced.

4.1 Cleaning and disinfection procedures

- 1) Remove the filter from the mask.
- 2) Administration of Detrisan or similar product inside the mask for 30 "
- 3) Administration of 5% sodium hypochlorite solution outside the mask for 5 '
- 4) Rinse the mask and submerge it into a water tank
- 5) Leave it to dry away from direct sunlight

5 Visual and functional check

Visual test

- 1) Inspect the mask to check there is no damage such as deformations, cracks, cuts in the facepiece, damaged valves.

The membranes of the valves, especially the membranes of the exhalation valves, are crucial elements for the functionality of the mask.

- 2) Defective or damaged parts must be replaced immediately.

• WARNING!

Do not use the mask if damaged

Functional test

After replacing the valves and / or the respiratory protection filter, repeat the sealing test.

6 Storage

Perform the cleaning and disinfection procedure.

Put the mask in its original packaging.

Keep it from being subjected to deformation.

Store it in a cool, dry place and protecting it from ultraviolet rays.

Keep it away from direct sources of light and heat.

7 Disposal

Dispose of the used respiratory protection mask and filter, in compliance with current waste disposal regulations.

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U.S.A. Branch SEAC USA Corp.